

Mortuary refrigerator 2 Body



MORTUARY REFRIGERATOR

BCCKH0417S

2 Berth for 2°C to 8°C

Dimension 870x2330x1900 mm

Net weight 230 kg

MORTUARY FREEZER

BCCKH0418S

2 Berth for -10°C to -20°C

Dimension 870x2330x1900 mm

Net weight 230 kg

the caster of refrigerator is fixed while transportation not to move.

When your side needs them to move without forklift, please adjust the caster to roll.

When your side install them at the final location, please have them fixed after installation while using.

How to operate the thermostat

1. Check that the power is 220V and plug in the cord.
2. Press the ON/OFF switch button (!). The current temperature is displayed on the digital screen
(In the case of initial operation, it starts after 1 minute)
3. When resetting the temperature, press the "SET" button for more than 1 second, then the set temperature will blink.
When it is blinking, press the button to set the desired temperature and press the "SET" button to save the newly set value.
4. Recommended operating temperature: Refrigerated 4°C,
Freezing - 10°C
(For the life of the machine, it is recommended to keep it operating at -10°C rather than using it for a long time at -20°C)